

Cleaning Guidelines for WoundPro™ and WoundPro+™ NPWT Systems

At Pensar Medical™, we prioritize the safety and effectiveness of our Negative Pressure Wound Therapy (NPWT) Systems. Proper cleaning is essential to ensure the longevity and reliability of your device. Please follow these guidelines to clean your unit effectively.



Utilize alcohol-based, hospital-grade quaternary disinfectant cleaners for optimum results.

01



Choose a Designated Area:

- + Perform cleaning of patient-care equipment in a designated area separate from clean or sterile supplies and food preparation zones.

02



Thoroughly Wash Your Hands:

- + Wash your hands thoroughly and regularly, especially after removing gloves.
- + Pay attention to clean around and under your fingernails and between your fingers.

03



Wear Safety Gear:

- + Wear protective gloves, protective apparel, and eye protection.

04



Disconnect All Components:

- + Remove the canister, tubing, and any dressing materials connected to the unit.
- + Dispose of these components following your facility's Biomedical Waste Disposal Policy.

05



Prepare Disinfectant Solution:

- + Prepare a detergent / disinfectant solution as per the manufacturer's instructions. ***WARNING: Do not mix detergent / disinfectants with other germicides or detergents.***
- + Utilize alcohol-based, hospital-grade quaternary disinfectant cleaners for optimum results.
- + Follow the manufacturer's instructions to ensure the disinfectant's maximum effectiveness.

06



Thoroughly Wipe Down the Unit:

- + Apply the alcohol-based cleaning solution onto a towel or wipe, then gently clean the unit's surfaces.
- + Thoroughly wipe down all surfaces of the unit.
- + Avoid using an oversaturated cloth.
- + ***WARNING: Never spray cleaning solutions directly onto the unit.***

07



Allow For Drying and Waiting Period:

- + Thoroughly dry the unit with a clean cloth.
- + Allow the unit to air dry for at least 30 minutes.

08



Properly Dispose of Bodily Substances:

- + Properly dispose of all bodily substances as directed by your facility's Biomedical Waste Disposal Policy.

09



Properly Remove Safety Gear:

- + Remove and dispose of gloves, protective apparel, and eye protection following facility protocols.

WARNING:

Direct contact with bodily fluids is potentially dangerous and may result in severe illness or death. Always observe universal precautions when handling contents from the WoundPro™ and WoundPro+™ canisters, tubings, or dressings.

CRITICAL WARNINGS

- ⊗ Do Not Mix Detergent/Disinfectants With Other Germicides Or Detergents.
- ⊗ Do Not Directly Spray Liquids Onto The Unit.
- ⊗ Do Not Heat Or Steam Autoclave The Control Unit.
- ⊗ Forceful Spraying May Lead To Ingress And Damage To Internal Components, Affecting The Device's Functionality.



Fluid Containment and Dressing Safety

The WoundPro™ and WoundPro+™ NPWT Systems are designed to contain bodily fluids within the tubing and canister. Take precautions to protect yourself from exposure to bodily fluids and follow your facility's protocols for cleaning medical devices.

It is essential to also refer to the instructions included with each dressing. Review the user manual for more information about dressing changes.

Disclaimer: This pamphlet is for educational purposes only and does not substitute for professional medical advice or specific guidelines provided by your facility. Always consult with your healthcare team for patient-specific instructions and protocols.

Contact Us

For any questions or concerns regarding the cleaning of your WoundPro™ or WoundPro+™ Systems, please refer to the user manual or contact our customer support team. Your patient's well-being is our priority, and proper maintenance ensures optimal performance and safety.